

6/01/05

Bed End Safety Pins

We are now providing bed end locking pins with Whisper Lite II beds. These pins are not necessary for normal bed operation, however they will prevent the bed ends from inadvertently rolling off when moving or pushing the bed or while installing or using a bed mount trapeze.

Follow instructions for bed assembly in your Whisper Lite user manual.

After the bed ends have been attached, insert the clevis pins and washers into the frame rivet plates and secure with the cotter pins. The flat washer should be on the outside of the bed end hook.

Bed End Safety Pin Kit 650-1000-000

4 clevis pins 100-7925-006

4 hitch pins 100-2001-006

4 washers 100-8625-002

