

⚠ WARNING: Important! Read and understand these instructions before installing or using the GF6650A-1 Homecare Clamp-On Half Rail. If you do not understand any part of these warnings, cautions or instructions, contact a healthcare professional for direction in the use of this product. If the GF6650A-1 Homecare Clamp-On Half Rail is not properly installed and adjusted, personal injury and/or damage to the Homecare Clamp-On Half Rail could result.

For information on Bed Rail safety, please see the following brochure: "A Guide to Bed Safety: Bed Rails in Hospitals, Nursing Homes and Home Health Care: The Facts" on the FDA website, at <http://www.fda.gov/cdrh/beds/>

⚠ WARNING: The GF6650A-1 Homecare Clamp-On Half Rail is designed for use in the homecare setting.

⚠ WARNING: If components are damaged or missing, contact your dealer immediately. DO NOT use substitute parts. Use only Lumex replacement parts. Non-Lumex replacement parts could cause entrapment, personal injury, and/or damage to the Homecare Clamp-On Half Rail.


⚠ WARNING: The GF6650A-1 Homecare Clamp-On Half Rail has no weight limitation when used with a manual or electric bed. The Homecare Clamp-On Half Rail is intended to define the edge of the sleep surface and to help prevent an individual from inadvertently rolling or climbing out of bed. Do not use the Homecare Clamp-On Half Rail as a restraint. If an individual is capable of injuring himself/herself, consult a physician for alternative means of safe restraint.

⚠ WARNING: GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper installation or use of this product.


INSTALLATION

Movable clamp disassembly

Homecare Clamp-On Half Rail, as shipped, is shown at near right. Before installation, you must disassemble the movable clamp.

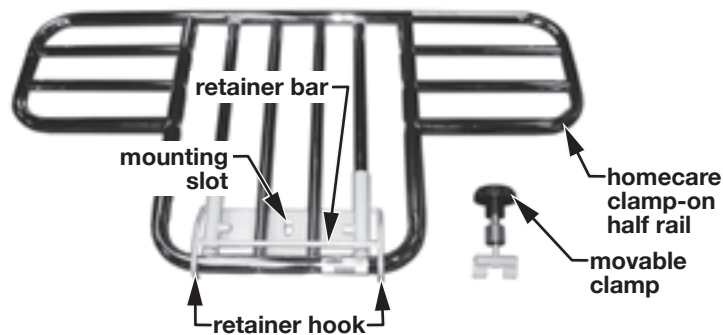

homecare clamp-on half rail and movable clamp as shipped

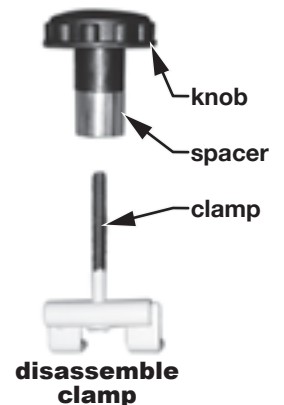


Unscrew the knob from the movable clamp assembly to separate the knob, spacer, and threaded clamp, as shown at far right.


Homecare Clamp-On Half Rail installation

Note: Homecare Clamp-On Half Rail may be installed on either head deck angle or foot deck angle. The Homecare Clamp-On Half Rail CANNOT be installed on pan deck.


1. Attach movable clamp jaw over deck angle, with threaded rod facing outward, as shown at near right.
2. Position half rail with fixed clamp jaw toward bed and rails facing outward, as shown at far right. Position half rail mounting slot over movable clamp threaded rod and slide until fixed clamp jaw is installed under deck angle.
3. Install spacer over end of movable clamp threaded rod and slide it against half rail flat surface, as shown at near right.
4. Install knob on end of movable clamp threaded rod, as shown at far right. Screw on until the clamp that secures the half rail to the bed frame is tight.

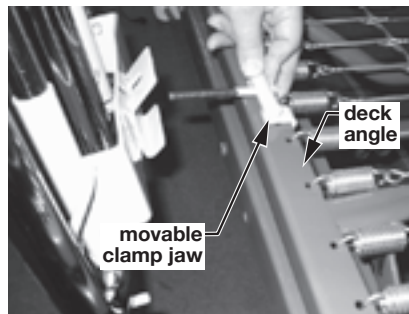


install movable clamp jaw over deck angle

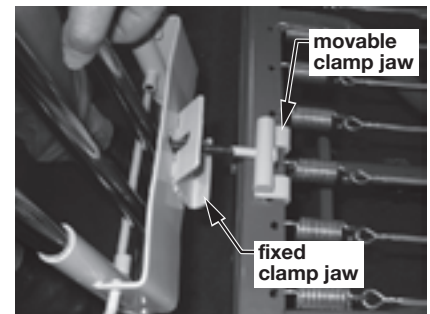

install fixed clamp jaw under deck angle

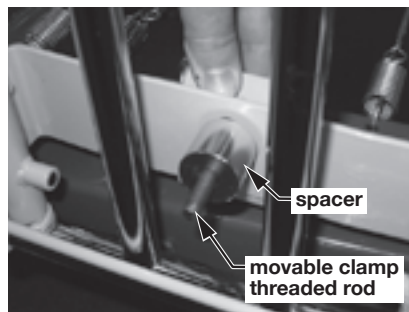

replace spacer on movable clamp

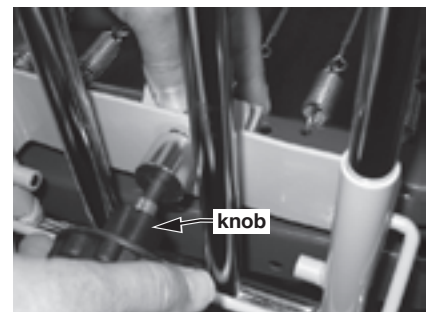

replace knob on movable clamp

RAISE AND LOWER HOMECARE CLAMP-ON HALF RAIL

See Homecare Clamp-On Half Rail picture on previous page. **To raise half rail:** Pull out retainer bar, raise rail to highest position, and release retainer bar, ensuring that rail is securely trapped by retainer hooks. **To lower half rail:** Raise rail slightly to release retainer hooks, pull out retainer bar, lower rail, and release retainer bar.

⚠ WARNING: Before use, ensure that the Homecare Clamp-On Half Rail is assembled and installed as described, that all attaching hardware is securely tightened, and that the Homecare Clamp-On Half Rail is securely in place; inspect frequently to ensure that all attaching hardware is securely tightened, and that the Homecare Clamp-On Half Rail is securely in place.

⚠ WARNING: Do not use the Homecare Clamp-On Half Rail as a handle for getting into or out of bed; this could result in personal injury and damage to the Homecare Clamp-On Half Rail.

▲ Caution: Do not exert side pressure on the Homecare Clamp-On Half Rail; this could deform or break it.

▲ Caution: Do not use the Homecare Clamp-On Half Rail as a push handle for moving the bed; this could deform or break it.

