

OPERATION OF DUAL MASSAGE MOTORS

- Press the “**ON/OFF**” button to turn massage motors on.
- Press the “**CONSTANT**” button to operate in the steady massage mode.
 - In this mode, there is no pulse or wave action. The motors simply operate at a steady pace with no variation.
 - Press the “**INTENSITY**” button to change the intensity of the massage
- Press the “**WAVE**” button to operate in the wave massage mode.
 - In this mode, the head and foot massage motors operate in “waves” that alternate between the head and foot areas of the bed.
 - Press the “**SPEED**” button to change how fast or slow the “wave” action is distributed.
 - Press the “**INTENSITY**” button to change the intensity of the “wave” action.
- Press the “**PULSE**” button to operate in the pulse massage mode.
 - In this mode, the head and foot massage motors operate in short, quick bursts that alternate between the head and foot areas of the bed
 - Press the “**SPEED**” button to change how fast or slow the “pulse” action is distributed.
 - Press the “**INTENSITY**” button to change the intensity of the “pulse” action.

NOTE

- When you press a mode button (constant, wave or pulse) the first time, both the head and foot motors will activate.
 - Press the mode button a second time and only the head motor will operate
 - Press the mode button a third time and only the foot motor will operate
- When you turn the massage motors off, the system will remember what setting you had in place. Therefore, the next time you turn on the massage system, it will begin operation using the settings you had in place during your last session.