

- ⚠ **WARNING:** Important! Read and understand these instructions before assembling or using the GF8900-1 Overbed Table. If you do not understand any part of these warnings, cautions or instructions, contact a healthcare professional for direction in the use of this product. If the GF8900-1 Overbed Table is not properly assembled and adjusted, personal injury and/or damage to the Overbed Table could result.
- ⚠ **WARNING:** If components are damaged or missing, contact your dealer immediately. DO NOT use substitute parts. Use only Lumex replacement parts. Non-Lumex replacement parts could cause personal injury and/or damage to overbed table.
- ⚠ **WARNING:** GF Health Products, Inc. assumes no responsibility for any damage or injury caused by improper assembly or use of this product.

Assembly instructions (see picture at right)

1. Lay the table top upside down on the floor. Position the mast above it as shown at right. Install the six screws that secure the mast to the table top. Tighten screws.
2. Place the base upside down above the mast. Install the two washers, inserts and bolts that secure the base to the mast. Tighten bolts.
3. Install the four casters as shown at right.
4. Turn the table upright and place it on the floor. Remove the one cross recess head screw shown at right to adjust table height.

To raise the table, apply light pressure upward at any point on underside of table top.

To lower table, squeeze adjustment plate up towards table top and push table top down to desired height.

